


TheSupport *YouNeed* by HULALA HOME

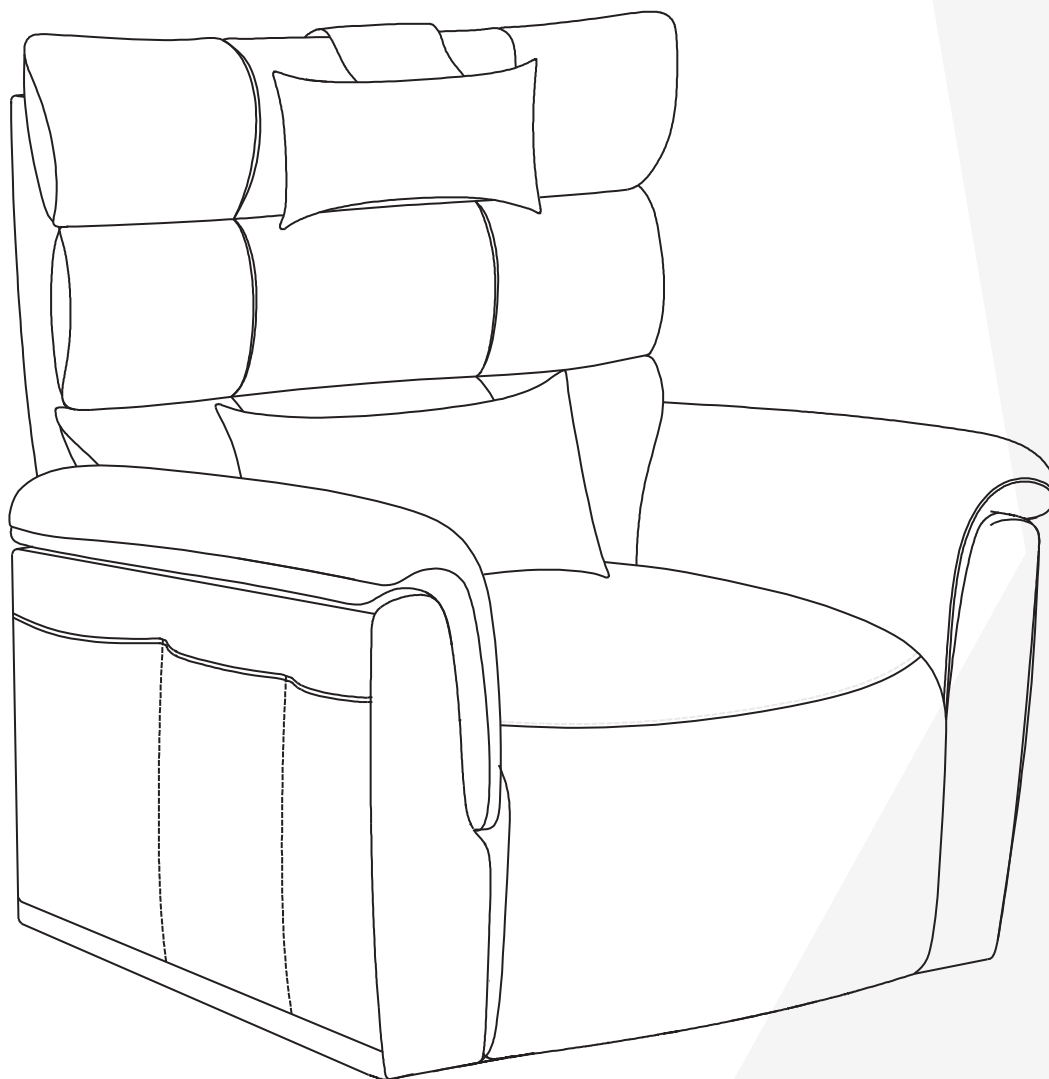

Weight Limitation
350 LBS



ITEM: RCLG1364

Embrace Power Lift Recliner

CONTENT

INTRODUCTION	01
SAFETY	02
SPECIFICATIONS	05
PARTS LIST	06
INSTALLATION	07
CARE AND MAINTENANCE	17
WARRANTY	18

INTRODUCTION

Welcome to the Hulala Home family! We are delighted to be part of your journey toward filling your home with comfort and care.

As a proud sub-brand of Hulala Home, TSYN is on a mission to redefine home comfort. Inspired by medical devices, we create furniture that blends thoughtful design with a focus on health and well-being, offering personalized solutions for every stage of life.

We ensure that each TSYN piece is crafted to enhance your quality of life, providing dignity, comfort, and care. With TSYN, your home becomes more than a space—it's a sanctuary of warmth and support, where every day feels better than the last.

We kindly encourage you to take a moment to review this manual, where you'll find valuable guidance on installation, operation, and maintenance. Your comfort and satisfaction mean the world to us, and we are always here to support you every step of the way. Let's begin this journey together!

If you have any questions or concerns, don't hesitate to reach out to us.

Call us: +1(833)324-2464

Email: customerservice@hulalahome.com

SAFETY

⚠ MANDATORY

Please read this manual carefully before installing or operating your power lift recliner.

Read and follow all instructions, warnings, and notes in this manual and all other accompanying literature before attempting to operate this product for the first time. If there is any information in this manual which you do not understand, or if you require additional assistance for setup or operation, please contact us. Failure to follow the instructions, warnings, and notes in this manual and those located on your product can result in personal injury or product damage and will void the product warranty.

NOTE: If you ever lose or misplace your copy of this manual, please contact us.



**Read and Follow
Owner's Manual!**



**Pinch/Crush Point
Hazard!**



Fire Hazard!



**Transformer Crush
Hazard!**



Child Crush Hazard!



Adult Tip Over Hazard!







Pet Crush Hazard!






**Child and/or Pet Tip
Over Hazard!**



⚠ WARNING


The following guidelines are provided to ensure the safe operation of your power lift recliner.


- Care should be taken in the unpacking of this unit, so as not to cause injury to persons or damage to the unit. Dispose of all plastic bags and covers, as they can be a smothering hazard to small children.
- To operate this recliner properly, you must read and understand the intended use of the product, and overall operation of the recliner as misuse, or lack of proper training may lead to serious injury.
-  Do not allow children or pets near the recliner while the recliner is operating.
-  Do not allow children to play with or operate the recliner at any time.
-  Hand control should be unplugged and placed out of the reach of children when the recliner is not in use.
- Do not flop down into the recliner when getting into it, or while it is in its lift position. Flopping down into the recliner will damage it and could cause personal injury.
- Do not allow anyone to sit on your lap, or on the arm of the recliner, or on the footrest while in or operating the recliner, as it may cause tipping and could result in personal injury.
-  Do not place fingers or hands between the seat cushion and the inside arm of the recliner while sitting in the recliner or when the recliner is in operation or plugged into a power source.


- 


Do not retrieve fallen objects from inside or under the recliner while the recliner is plugged into a power source.
- Use extreme care before operating your device to ensure electrical cables are not located near pinch points, such as in between the seat and arms near the recline mechanism.
- 



Do not operate the recliner if there are any persons, pets or other obstructions under or in front of the footrest, under the recliner or behind the recliner.
- 


Do not position yourself, child and/or pet too high on the recliner while the recliner back is reclined. This can cause a tip over hazard. Make sure you are properly fitted for the lift recliner prior to operating the recliner.
- 

Do not smoke while operating, sitting or resting in the recliner.
- 

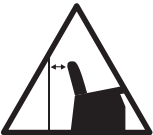To reduce the risk of electrical shock, avoid immersing electrical components in liquid or spilling liquids onto them. Do not attempt to open electrical components, as there are no serviceable parts inside.
- 

Warranty will be void if any of the electrical components are opened by other than a qualified service person.
- 

Make sure the recliner is placed at the specified distance from a wall/corner. Failure to do so can damage the recliner and will void the warranty.
- 

Not for commercial use, only for residential use.




SPECIFICATIONS

<i>Recliner Specifications</i>	
Weight Capacity	350LBS
Power Input	AC110-240V
Power Output	48W
Rated Voltage	DC24V
Rated Current	2A


<i>USB Specifications</i>	
Rated Voltage	5W
AMP	2A
Rated Wattage	10W

PARTS LIST


BOX A

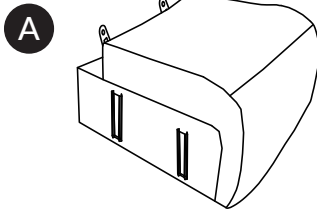


Seat Frame

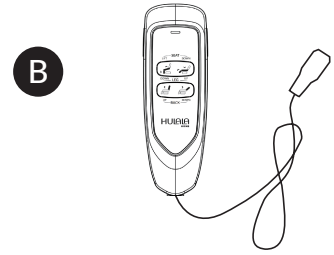


Remote Control

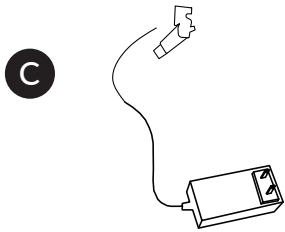

Power Source

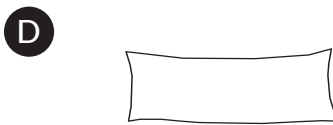


Lumbar Pillow

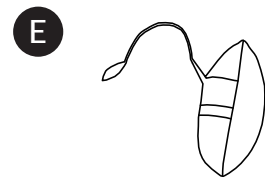


Adjustable Neck Pillow


BOX B

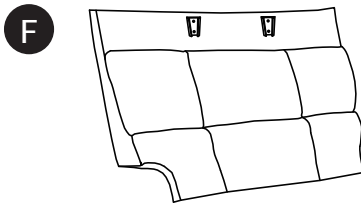


Backrest

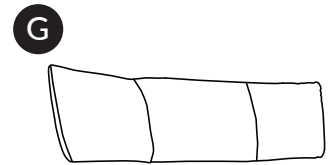

Active Headrest

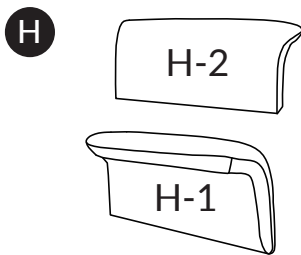


Soft Armrest Covers

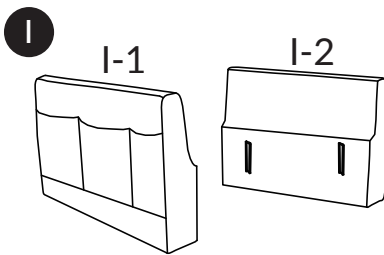


Armrests



Installation:
15 minutes.

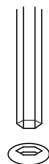


Required for
Installation

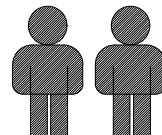

Two People



INSTALLATION

Step 1

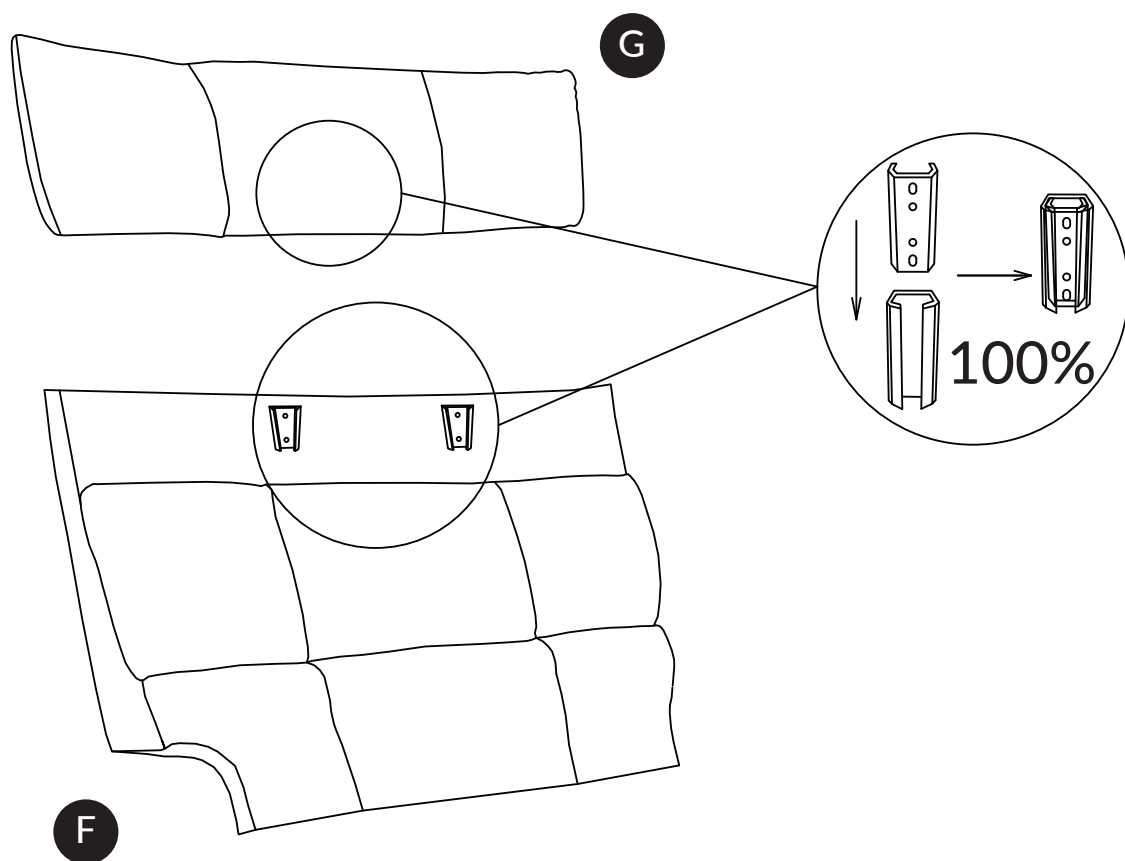

Step 2


Start by installing the zipper on the soft armrest covers.

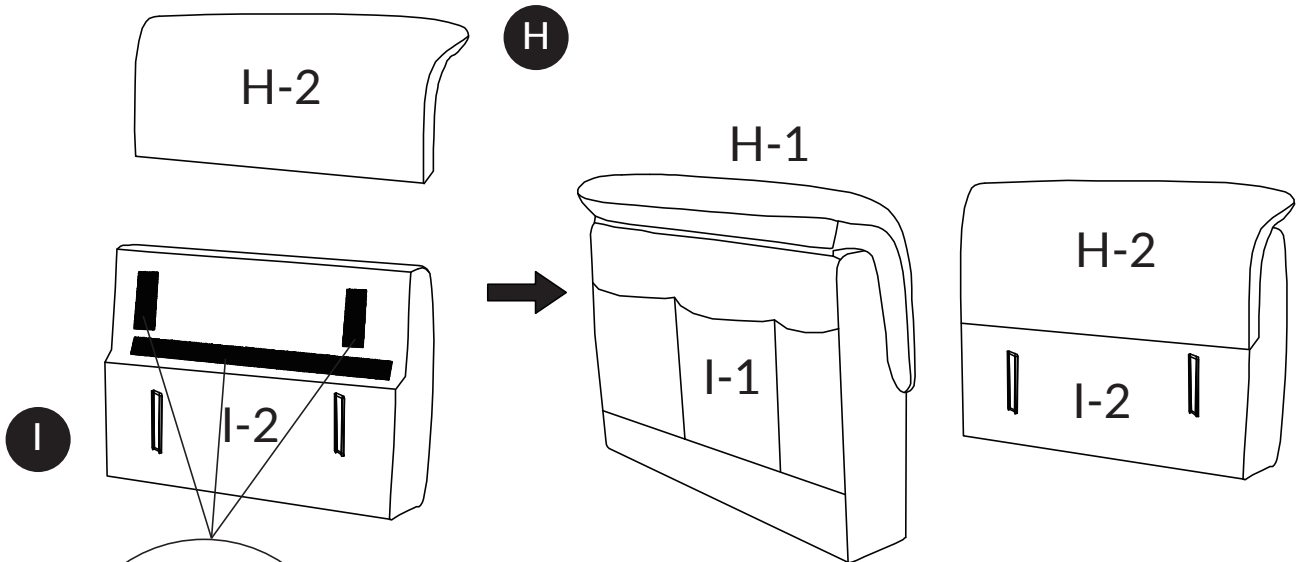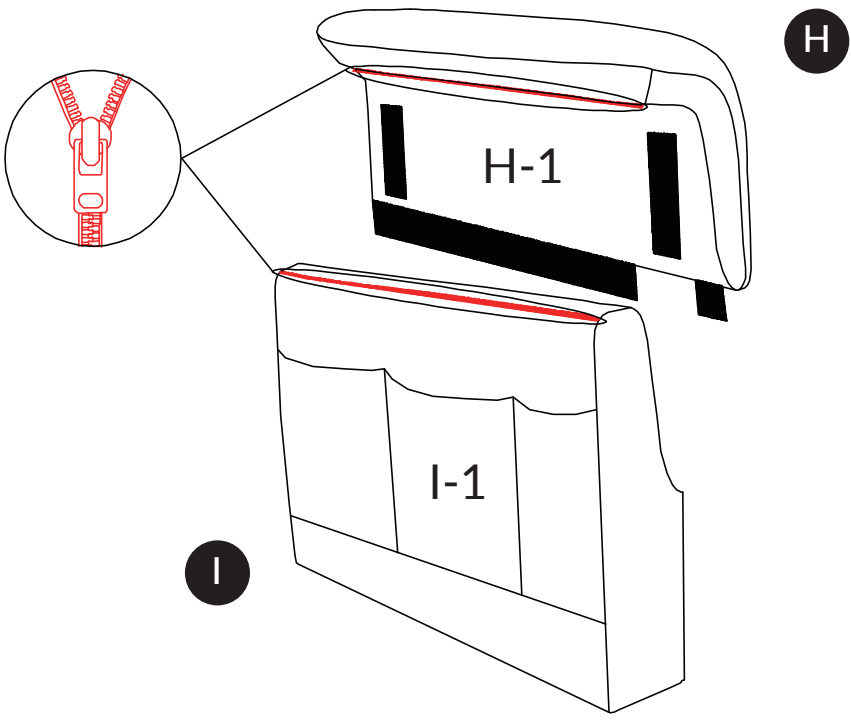

Remove adhesive strips, then attach armrest covers with Velcro.





Step 3

Insert the armrests into each side of the seat frame, ensuring the slots are fully aligned with the base.

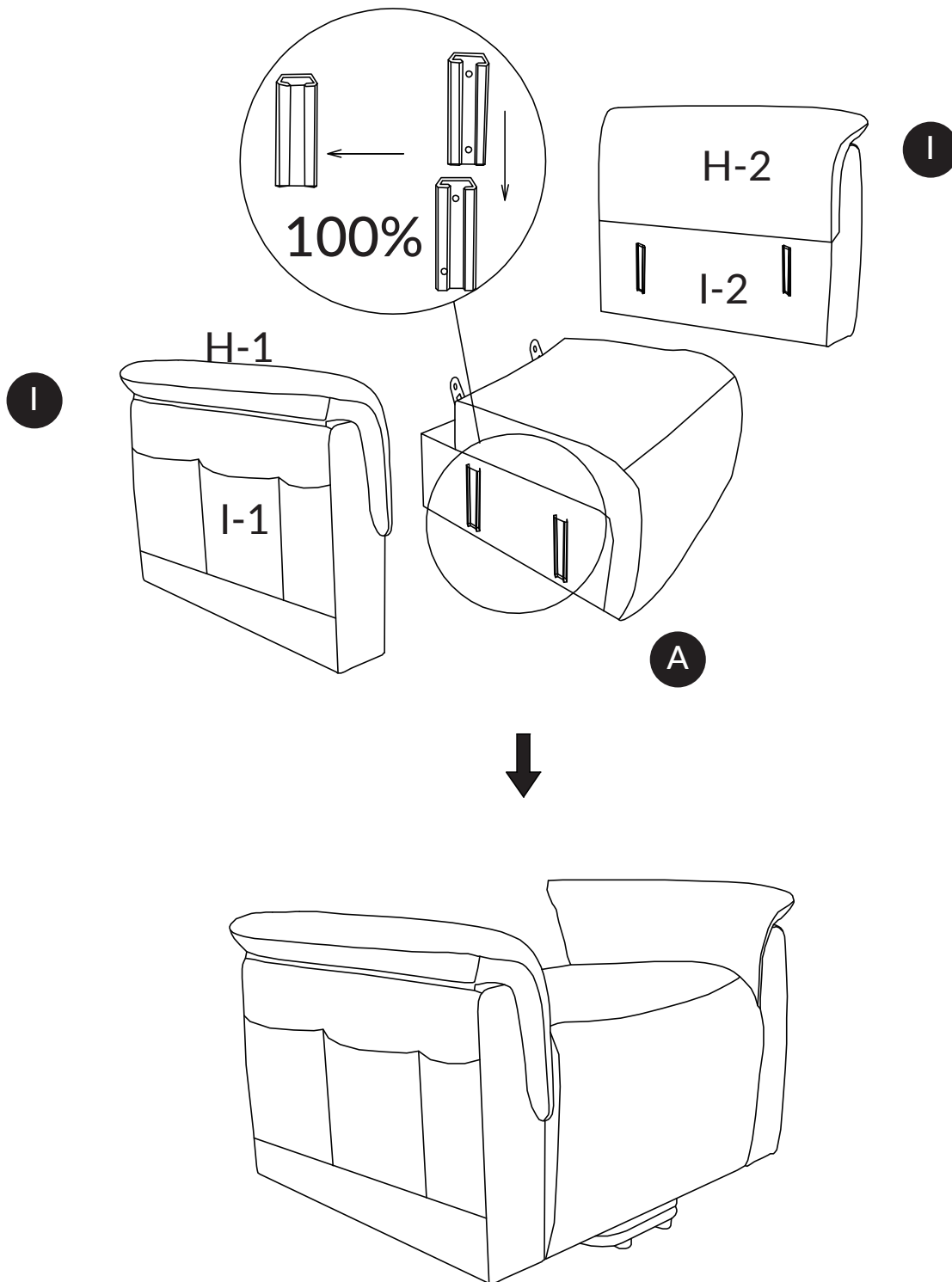




Step 4

Insert the backrest into the seat frame, ensuring the slots align perfectly with those in the base.

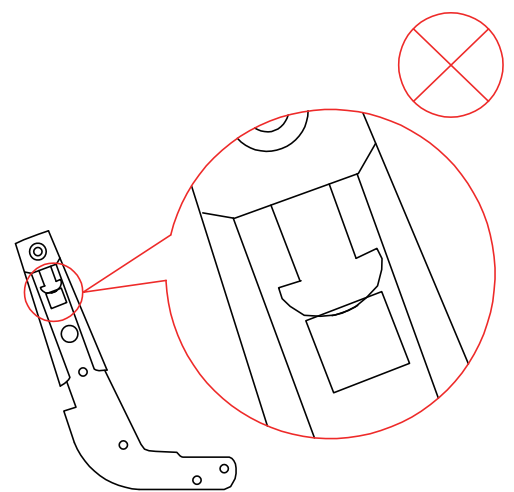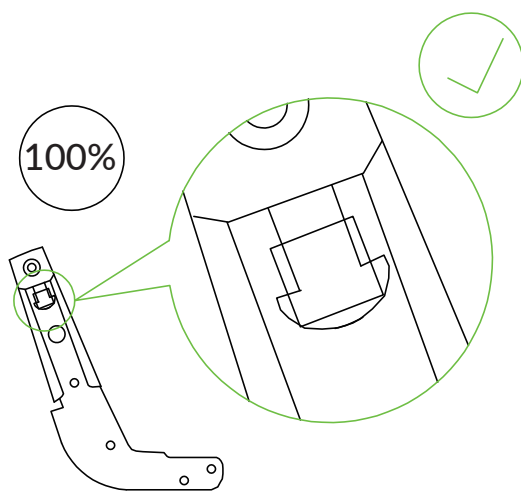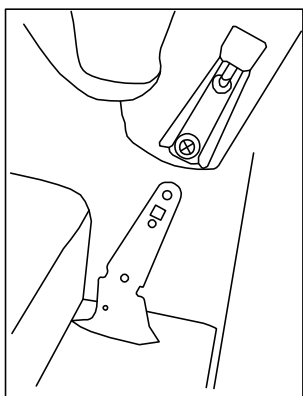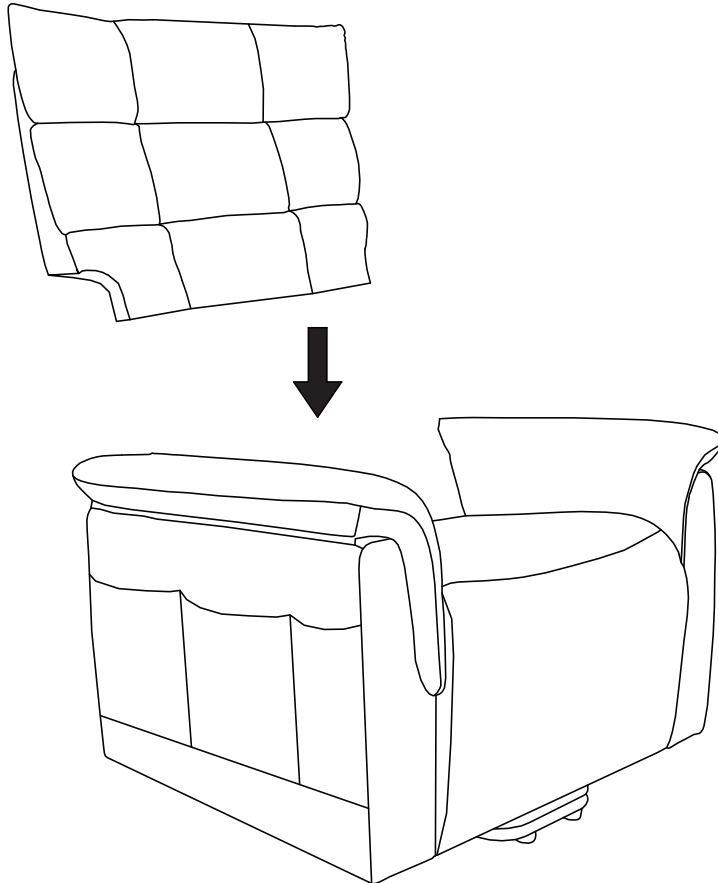


If the backrest needs to be removed, follow these steps:

1. Lift the cover beneath the backrest.
2. Locate the tabs on both brackets under the cover, one on each side of the backrest, and lift them.
3. Carefully lift the backrest upward to detach it from the seat.

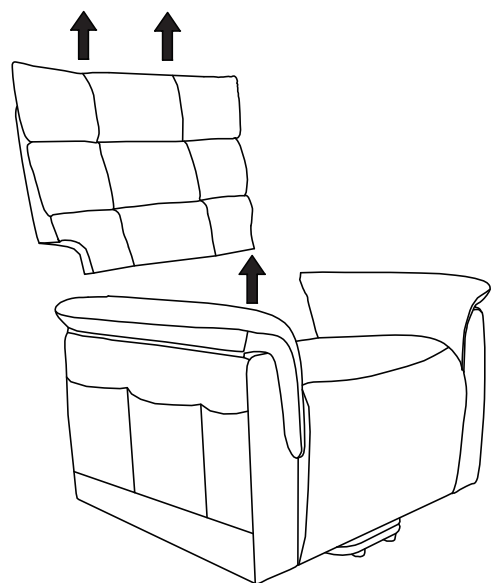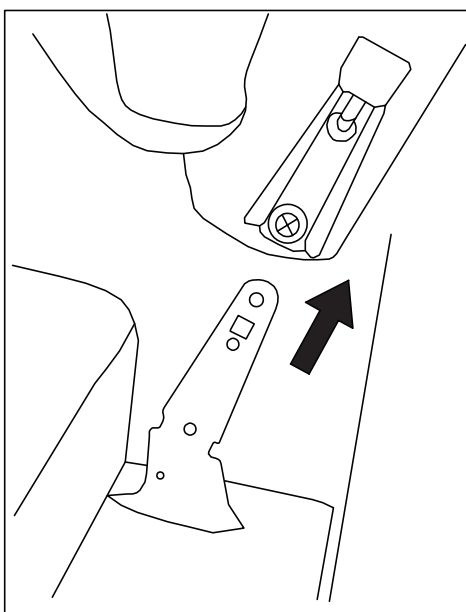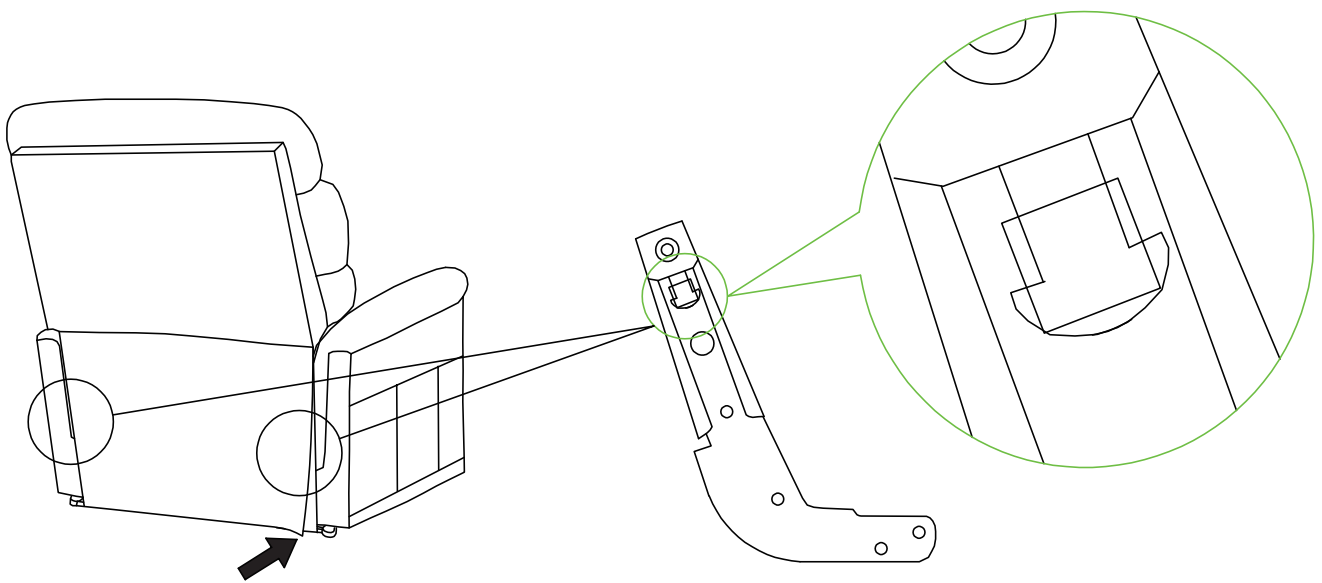



Step 5

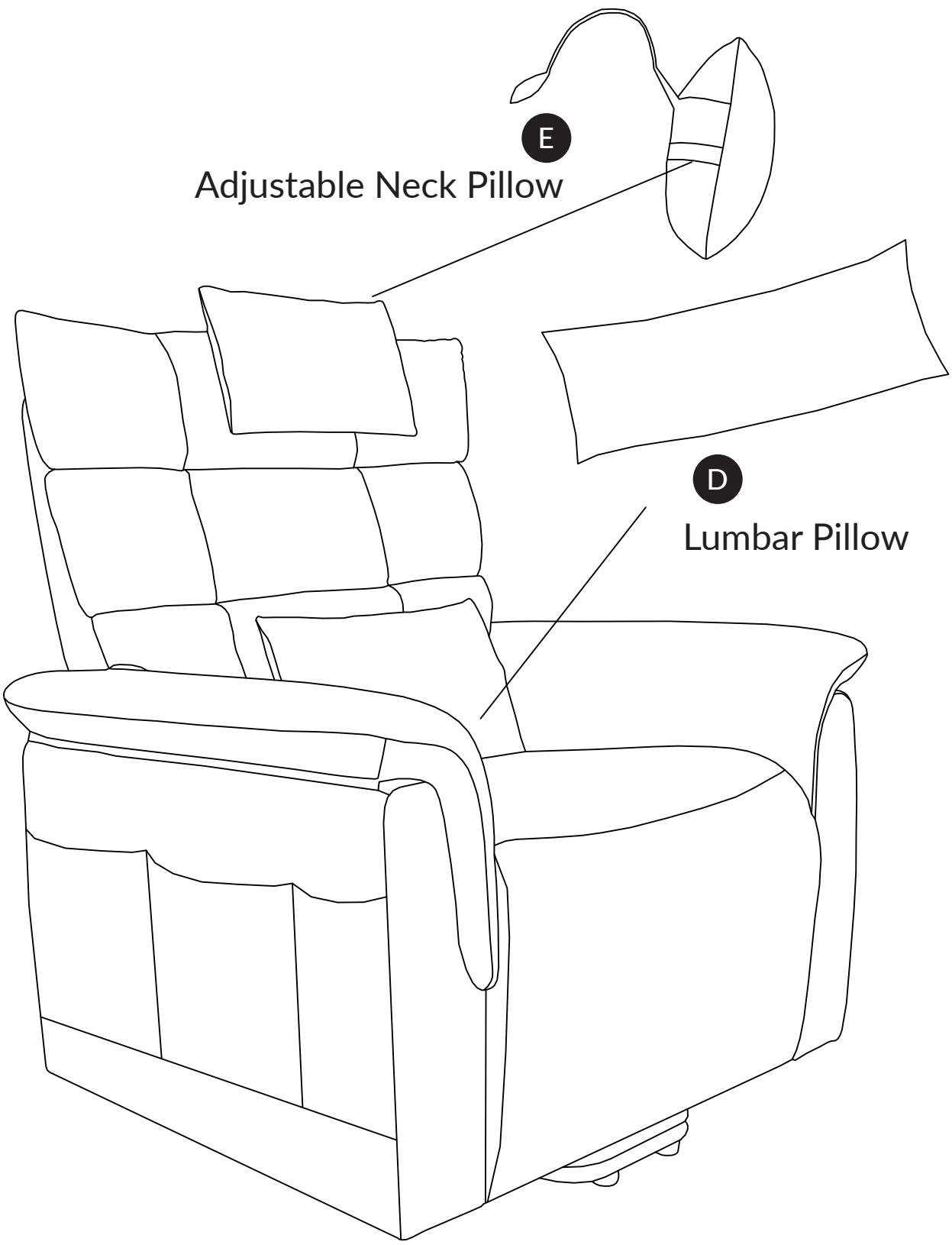

Adjustable Neck Pillow

Lumbar Pillow


Step 6

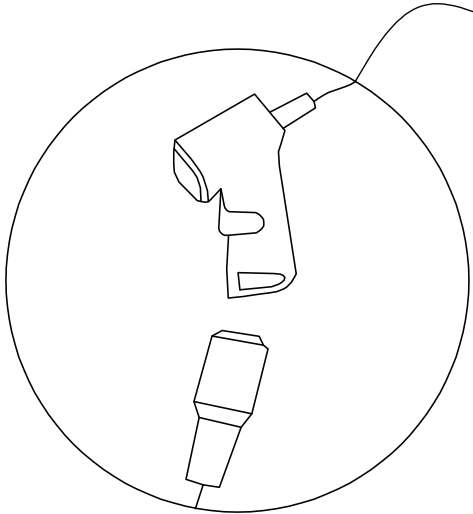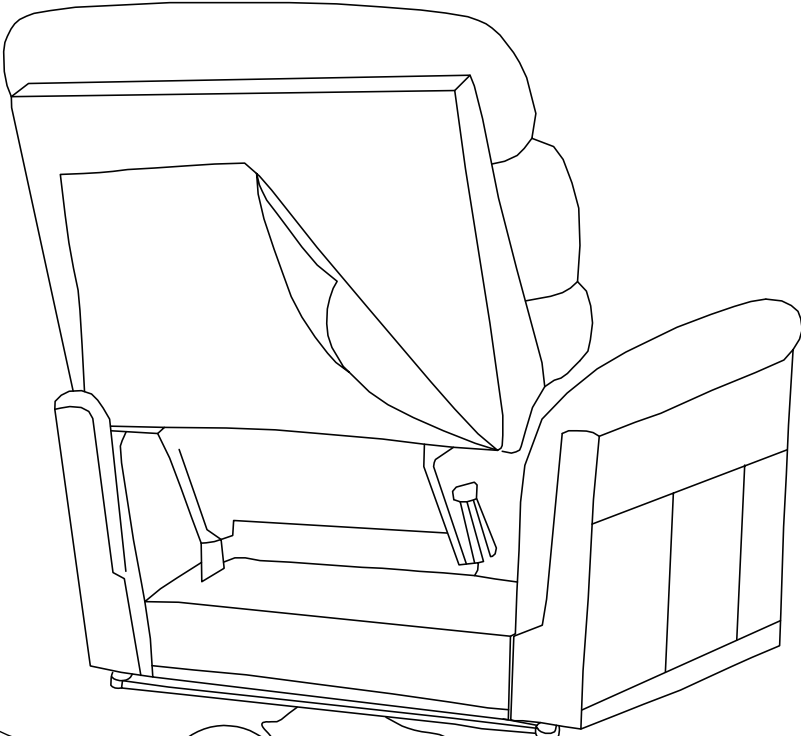

Wire #1 connects the power to the controller (B).

Power Source **C**

Remote Controller **B**

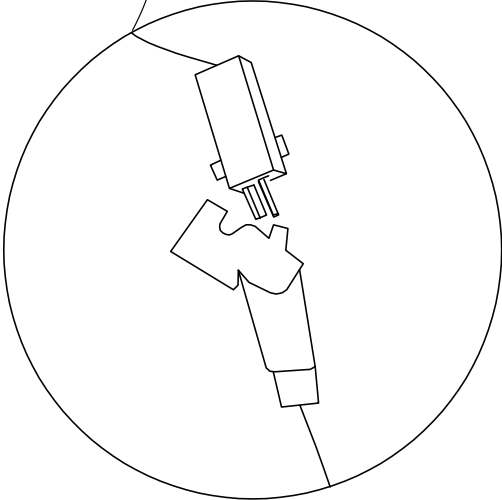


Wire #2 connects the recliner to the power source (C).


Step 7

Adjustable Lateral Support:


Provides head and lower back support during bending.

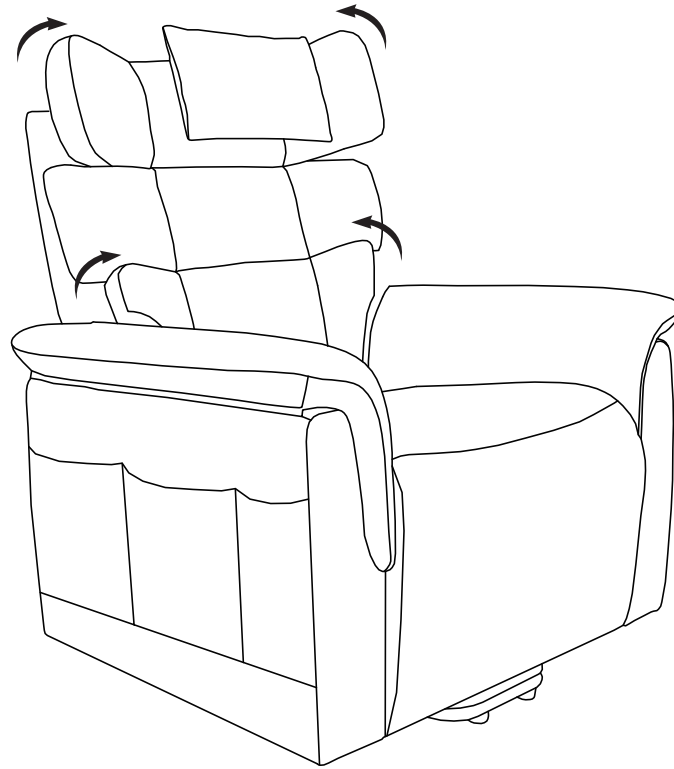


How to restore bilateral support:

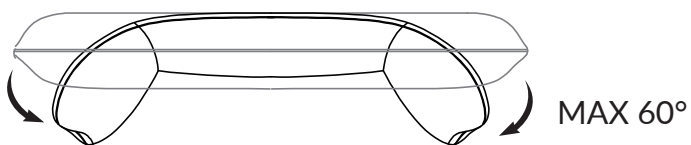

Step 1: Gently lift the lateral support until it reaches a maximum angle of 60 degrees.

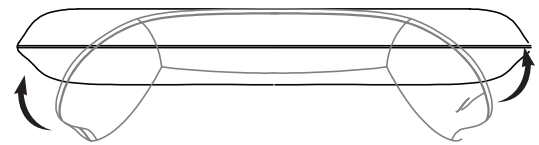


Step 2: Push the lateral support back into position, engaging the locking mechanism.

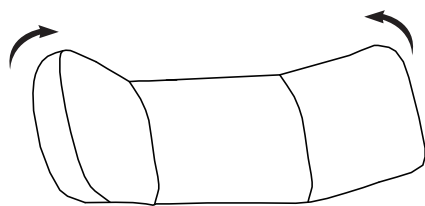

Step 3: Listen for a "click" sound to confirm it is securely locked and ready for use.

Step 8

Remote Control Instructions:

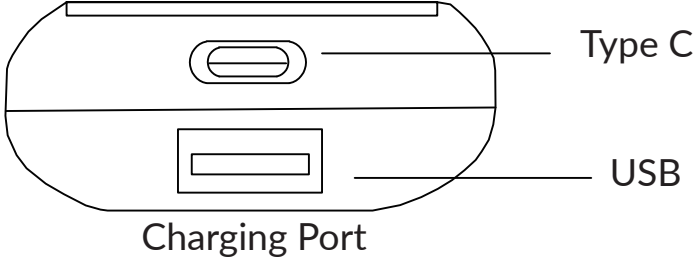


Press to retract the footrest and raise the seat.


Press to lower the seat and unfold the footrest.

Press to move the seat back forward.


Press to move the seat back backward.

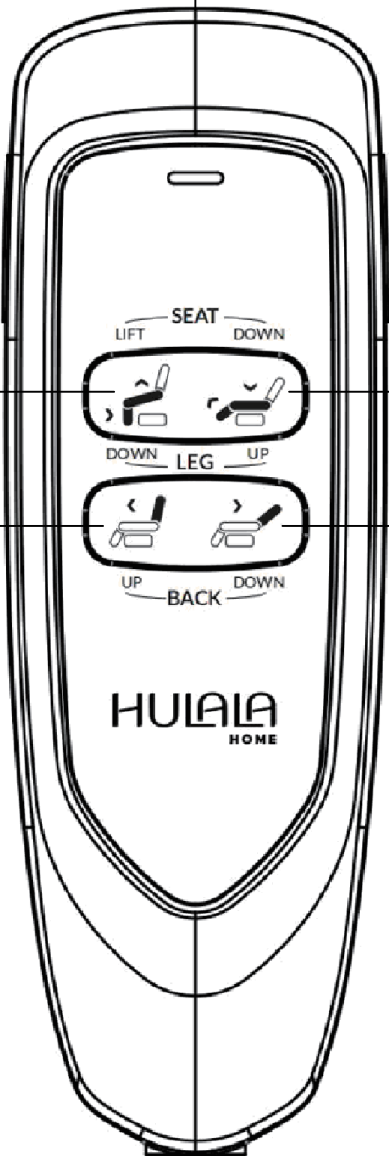

Step 9

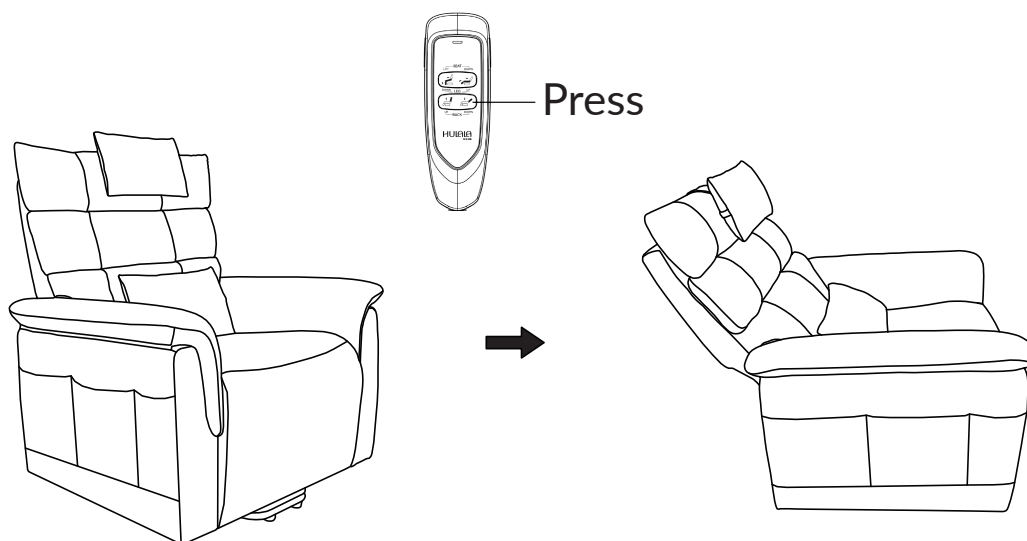


Press the button to recline the backrest.
Tip: Kindly begin by individually adjusting either the backrest or the footrest.

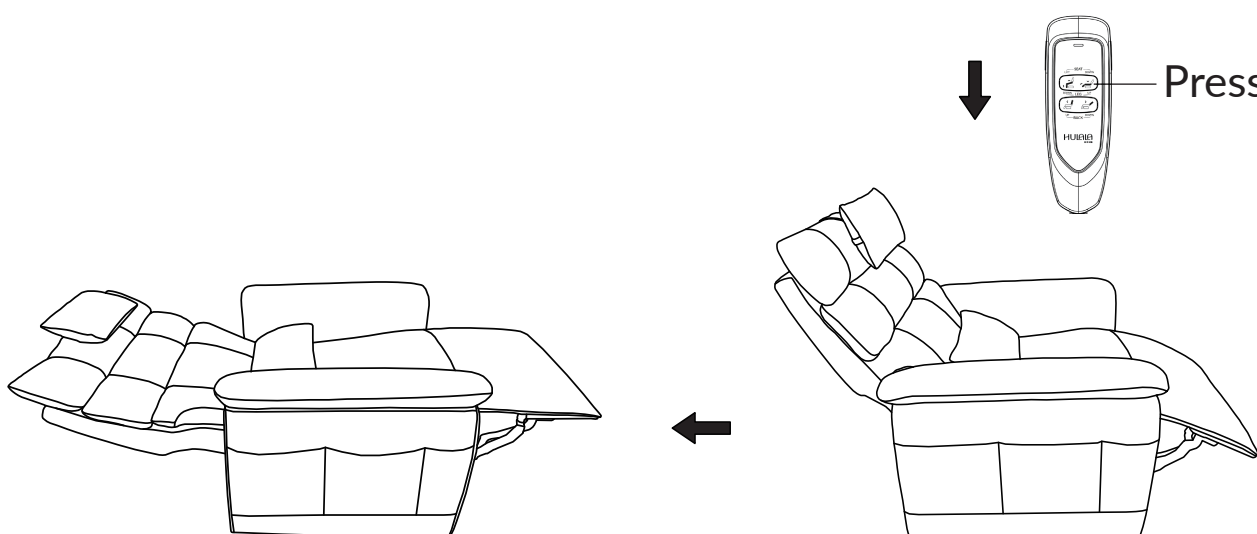


Please ensure you adjust your body position appropriately while the backrest is moving.

Press the button to raise the footstep.

Step 10

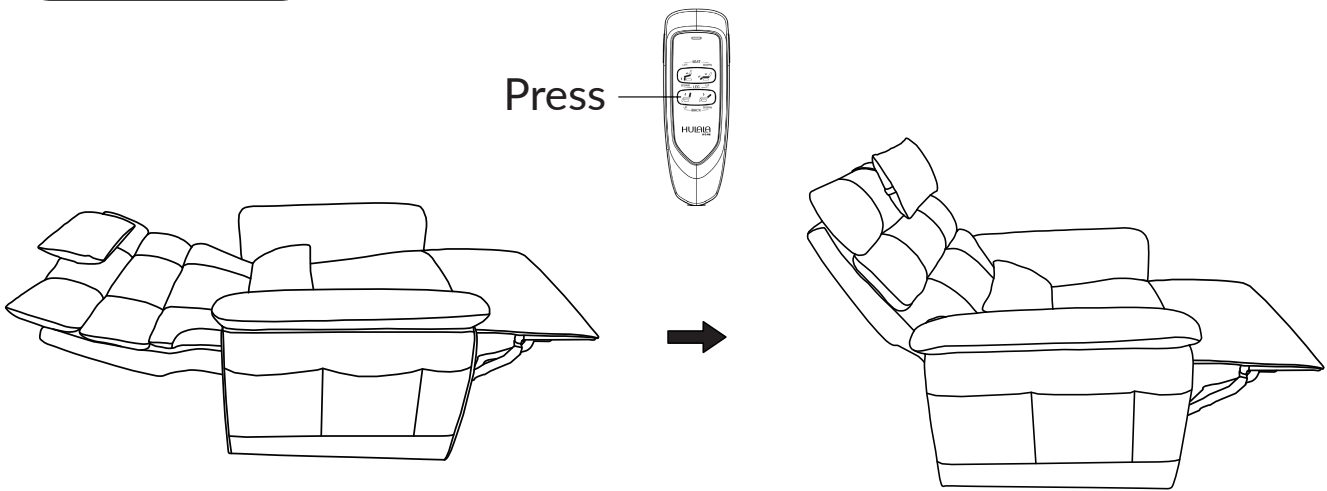

Press the button to adjust the backrest to its lift position.

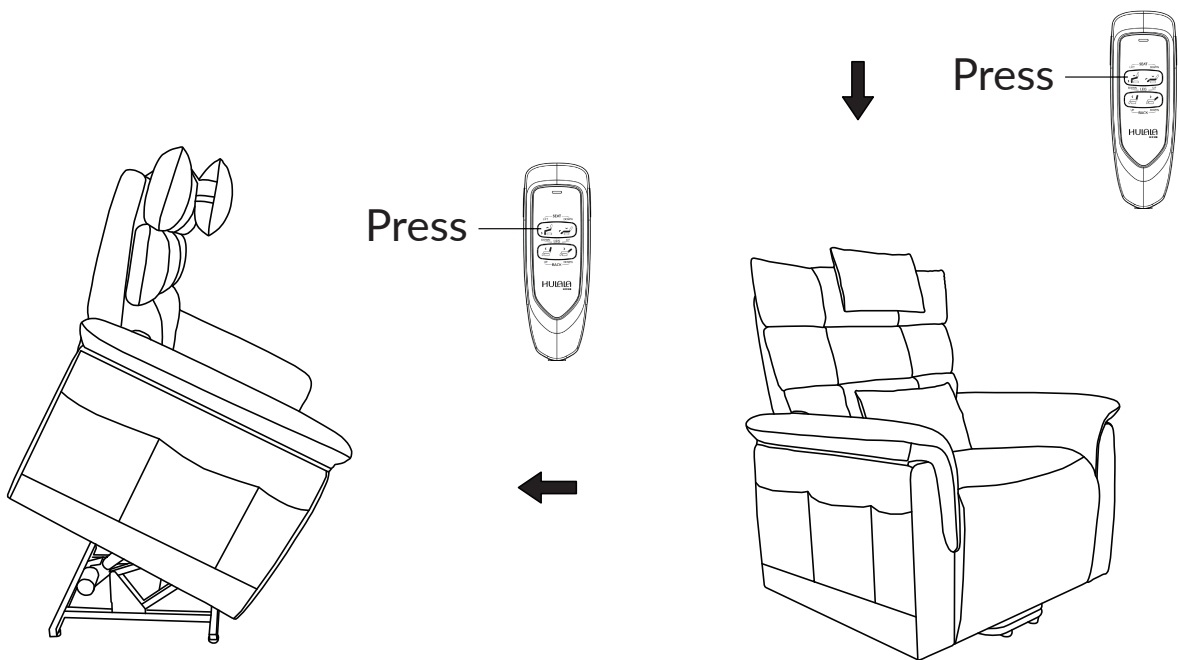

When returning the recliner to its original position, press and hold the button to adjust it back to the lift position.

Press the button to retract the footrest.



NOTE: Ensure the backrest is fully returned to its original position before adjusting to the lift position.

CARE AND MAINTENANCE

Fabric and Faux Leather Care Instructions

Cleaning

Do not remove cushion covers for cleaning or washing, even if they have zippers. Remove liquid spills immediately by blotting with a soft, dry, absorbent cloth or sponge. Clean by gently wiping with a soft cloth dampened with soapy water. Wipe away excess liquid with a soft, dry cloth. Allow the cleaned area to dry in the shade, away from direct heat, before use.

Do not rub excessively or use abrasive materials. Branded fabric cleaners may be used, provided the manufacturer's instructions are carefully followed. Avoid using abrasive powders, pastes, or alcohol-based products. On-site hot water extraction or dry-cleaning machines may be used with strict adherence to the machine instructions. Only use upholstery detergent. Damage resulting from incorrect use is not covered by the warranty. For persistent marks, seek professional advice. Re-application of soil-resistant treatments may be necessary after cleaning and is not covered by the warranty. Professional cleaning is highly recommended.

Spot Cleaning

Remove spills and stains as soon as possible. Gently scrape off any soil or blot up any liquid from the fabric surface. Apply proprietary cleaning agents strictly according to their instructions to remove spill residues. Do not rub excessively or use abrasive materials. Avoid saturating the fabric or interior cushioning with water or other cleaning liquids. Spot cleaning with a dry-cleaning solvent is possible, but always test on a hidden area first. For persistent marks, seek professional advice.

Pilling

Pilling can occasionally occur as a result of normal daily usage and should not be considered a fault. Fibre pills can be removed using a battery-operated pilling tool available from most haberdashery stores. Follow the manufacturer's instructions carefully to avoid damaging the fabric.

WARRANTY

- 1. We are committed to providing high-quality products and ensuring every customer is satisfied with our products and services.
- We offer a 30-day warranty from the date you receive your item. To process warranty claims, please provide proof of purchase, such as your order number or receipt. For items beyond the warranty period, we're still happy to assist you with additional support options. Please feel free to contact us!
- Replacement parts can be purchased directly from our company if they are available.

Are you experiencing difficulty with assembly or missing parts?

Please send an email with your order number to
customerservice@hulalahome.com.
Call us: +1(833)324-2464

For returns, please refer to the return policy of the retailer or marketplace where you made your purchase.

***We sincerely appreciate your purchase.
Brighten your life with our furniture!***